

# I-MED ONLINE IntelViewer for MACs

## Installation and Login Guide

### Installation

1. Select **Download IntelViewer for MAC**



I-MED Online IntelViewer will require installation on your device.  
Please click on the appropriate option below for installation.


> Download IntelViewer for PC

> Download IntelView for MAC



*If you do not have a username or password please complete the  
I- MED Online Registration Form on the left.*

2. Once downloaded open the installer. Drag IntelViewer into the applications folder

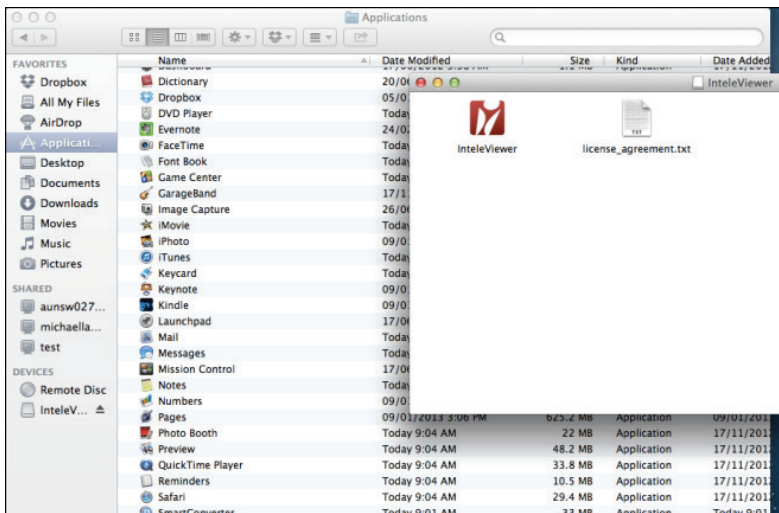


## Installation continued

- The first time you run the application you will be prompted to Enter a New Server  
Click **Enter a New Server**



- Type in the server URL  
**<https://pacs.i-med.com.au>**  
Click **Ok**

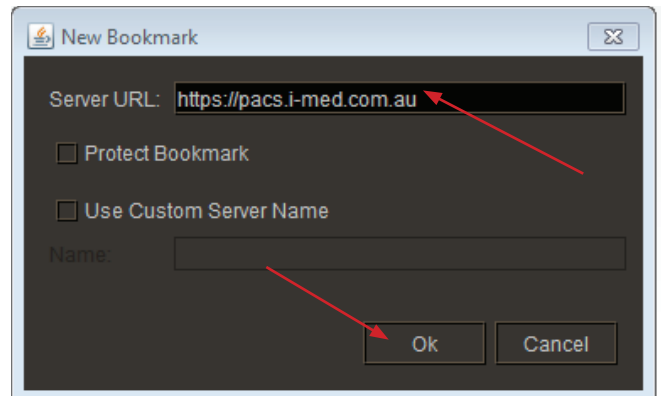

- Enter your username and password  
Click **Enter**

### TIPS:

Requires JAVA. If JAVA is not installed you will be prompted to install.

Apple's default security setting does not allow you to install applications from unverified sources. You can change this by going to: System Preferences/ Security & Privacy

